

Target 3D



Instructions for import of the implant library
Target 3D into DentalWings software



How to import the Target 3D Library


Import the library into the DWClient

1. Select Applications → Implant Kits

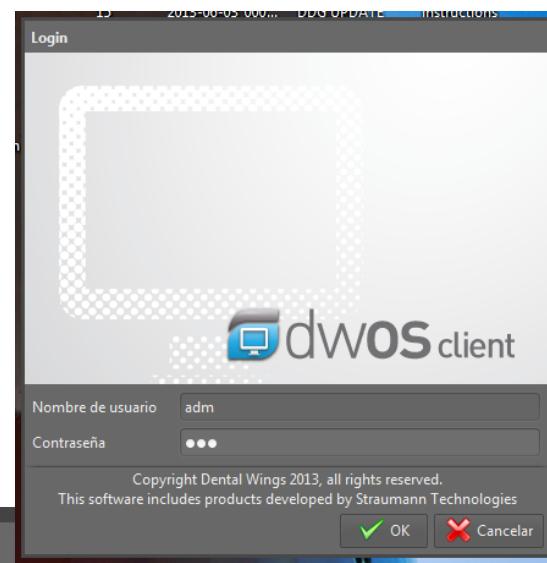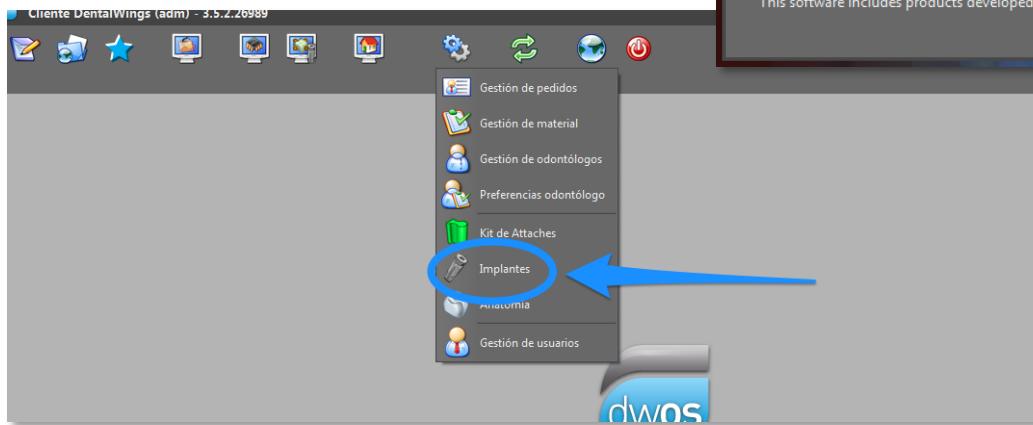

How to import the Target 3D Library

2. Right click on *Implant Kits* and choose *Import Implant Manufacturer Kits*

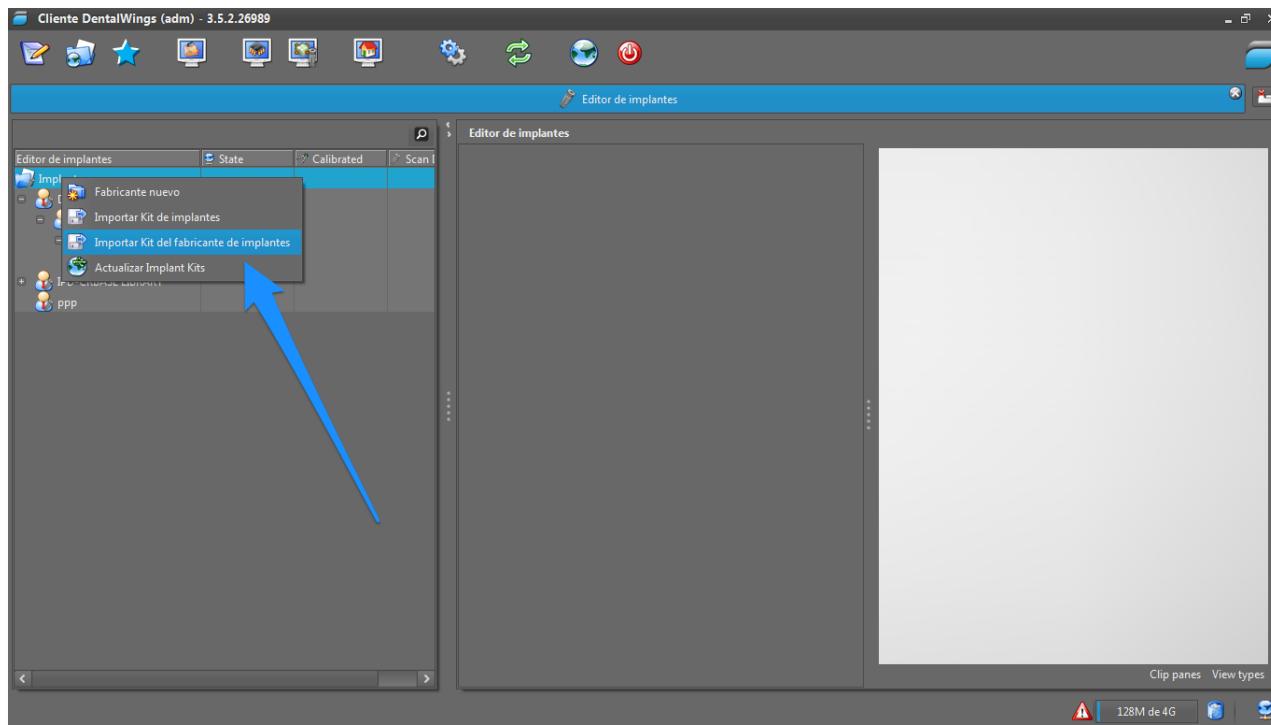


Target 3D



How to import the Target 3D Library

3. Identify the path of the Target 3D Library and choose open

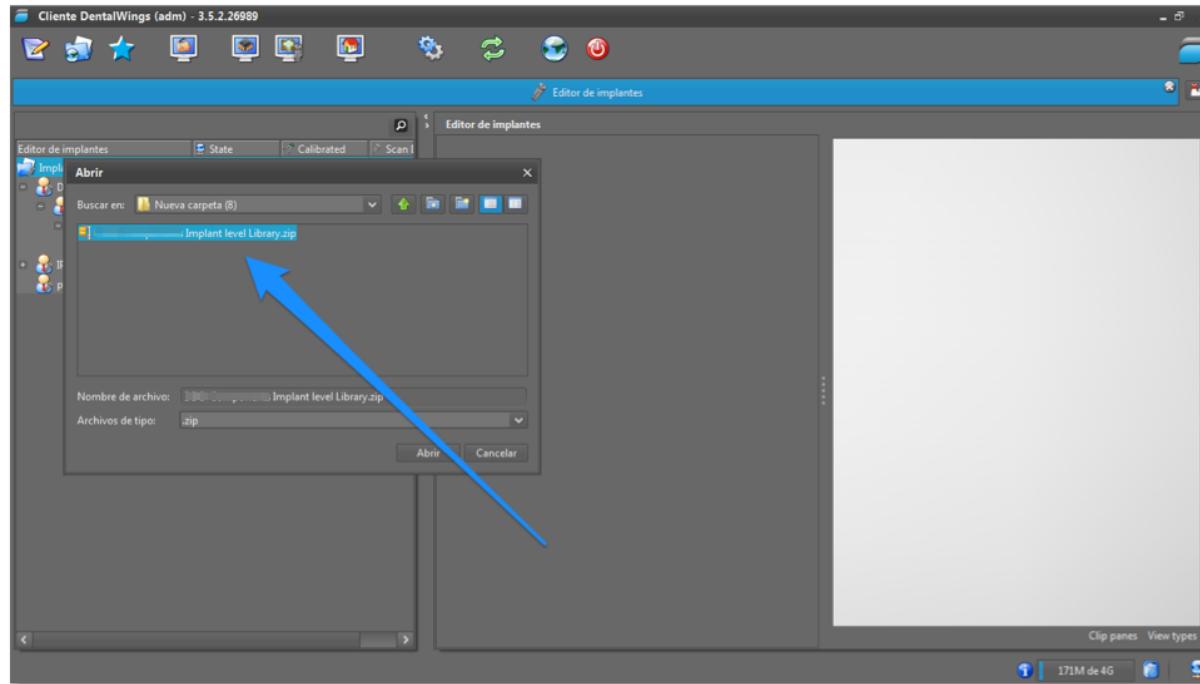


Target 3D



How to import the Target 3D Library

5. When the library is successfully imported, it will be added to the available implant Library list. All Kits of Target 3D Library will be integrated.

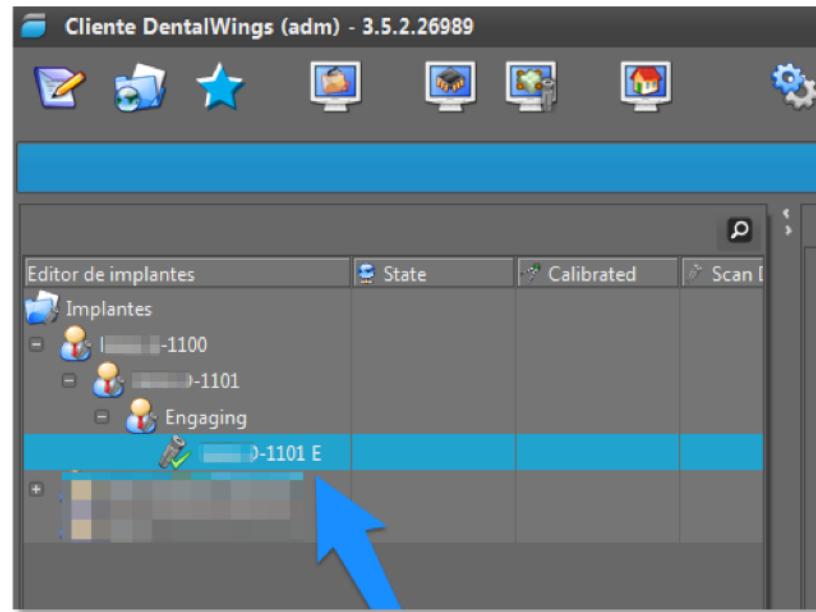


Target 3D



How to import the Target 3D Library

When you select a kit, displays a screen with all components of this Kit

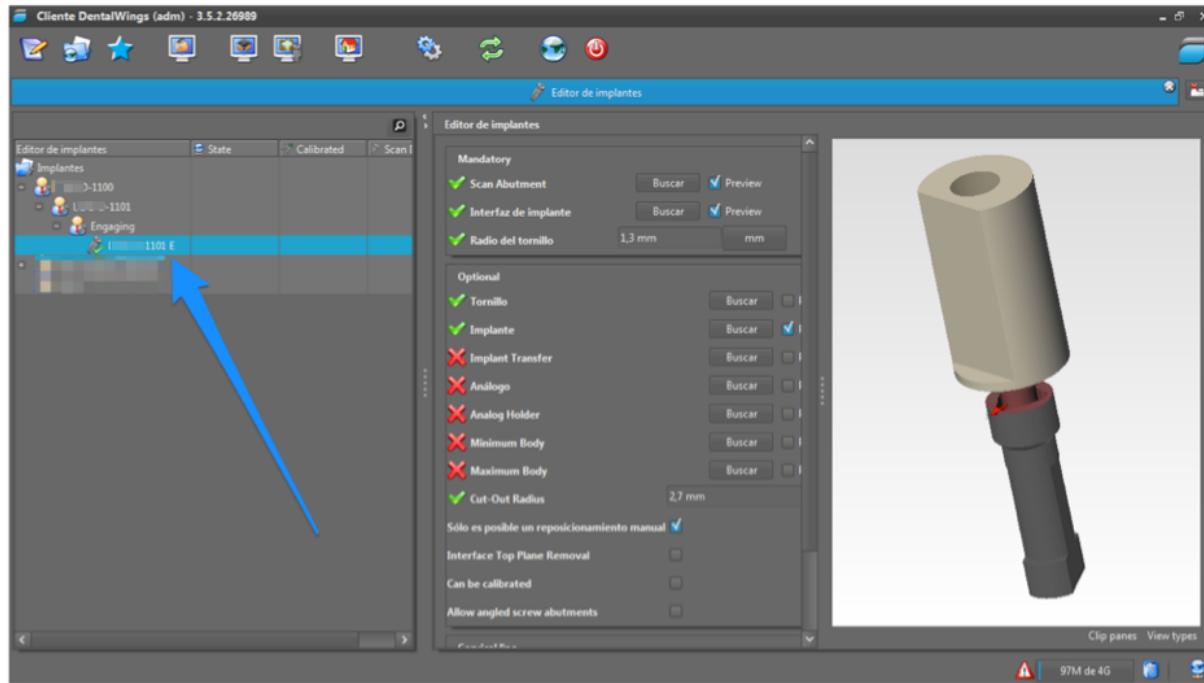


Target 3D



How to import the Target 3D Library

6. Now you can select the connection that you want to design

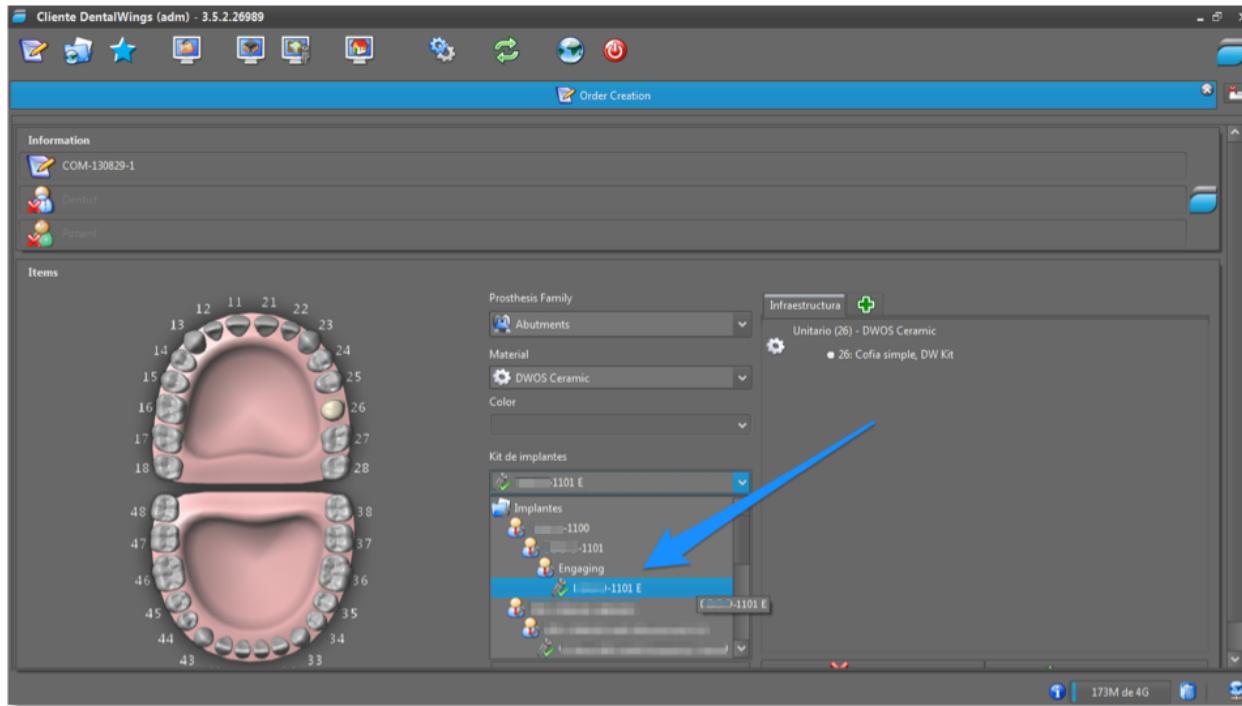


Target 3D



Target 3D Implant Library for DentalWings

Everything is ready to design this:

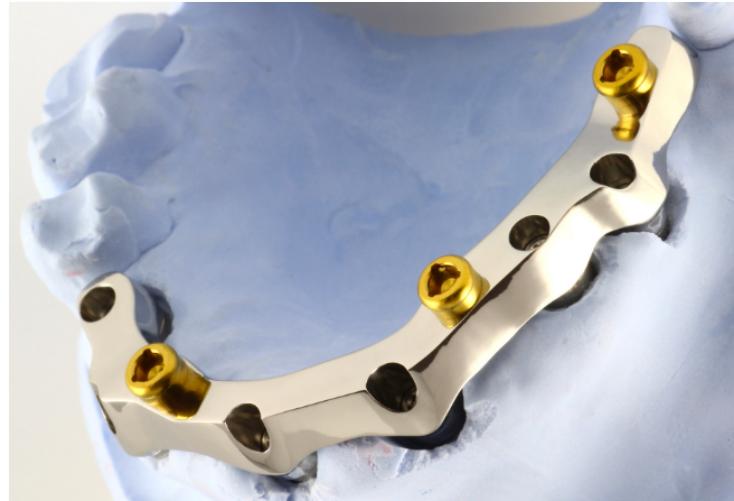

Target 3D



Thanks for Watching

Target 3D Support



Target 3D